

## INSTRUCTIONS FOR USE - TRANSPARENT, STERILE POLYURETHANE DRESSING WITH HYPOALLERGENIC LATEX-FREE ADHESIVE WITH COMFORT TECHNOLOGY

### Indications for Use

Indicated for covering specific topical areas of the body.

### General Description

Dressings. Sterile transparent polyurethane with a hypoallergenic, latex-free adhesive featuring Comfort Technology. The film includes a transparent, absorbent gel pad impregnated with 2% chlorhexidine gluconate with soft, non-woven fabric-reinforced edges, an application frame, two sterile strips, and a record label.

Available in different models, differentiated by coverage



### Size:

Code	Dressing size
060.088.0934	7X8.5 cm 2x2 cm
060.088.0942	8.5X11.5 cm 3X4 cm
060.088.0959	10X12 cm 3X4 cm
060.088.0967	10X15.5 cm 3X7 cm

### Instructions for Use

1. Open the package and remove the dressing.
2. Peel back the liner to expose the adhesive surface.
3. Place the dressing over the site and press to secure it.
4. Remove the frame or front liner while smoothing the edges.
5. Complete and seal the label, if included.

### General precautions

1. Do not use the product if the packaging is damaged, as this compromises its sterility.
2. Do not use this product if it has expired.
3. Do not reuse the dressing.
4. Do not re-sterilize the product.

