

Document Name: Instruction for Use

Document No.: IFU-GS-M020105

Product : Gauze Swabs

Version: 02



## Instruction for Use

### 【Product Name】

Gauze Swabs

### 【Product Composition】

The product is mainly composed of white or dyed bleached gauze, gauze is made of 100% cotton.

### 【Intended Use】

It's intended to be used as mechanical barrier, for compression or for absorption of exudates. This is a single use device.

### 【Intended User】

Medical staff or lay person

### 【Intended patient population】

Used for all age groups, not recommended for infants to use.

### 【Handling Instruction】

- Clean the wound area, hands and instruments in accordance with normal procedures.
- Check the package, if the package is not intact, or the product is abnormal compared with others, it must be replaced.
- Open the packaging and take out the product, gently place the product on the wound that needs to absorb exudates.

### 【Warnings and Precautions】

- The product is disposable and cannot be reused.
- Stop using if product has any defect.
- Stop using if the product has expired.

---

Document Name: Instruction for Use

Document No.: IFU-GS-M020105

Product : Gauze Swabs

Version: 02

---



**【Shelf Life】** 5years

**【Contraindications】**

Do not use on patients with a known hypersensitivity to the product itself or to its components.

**【Side Effects】**

None

**【Disposal】**

Disposal should be handled according to local environment procedures.

**【Special Storage Conditions and Methods】**

The product should be stored in a cool and dry place and avoid direct sunlight.

The transportation should prevent the weight, sun and rain.

**【Notice】**

Any serious adverse event related to this product must be report to Allmed Medical Products Co., Ltd or authorized representative or national health authority in your country.

Document Name: Instruction for Use

Document No.: IFU-GS-M020105

Product : Gauze Swabs

Version: 02



### 【Explanation of figures, symbols and abbreviations used in labels】

|  |                                  |  |                         |  |                                                   |  |                              |
|--|----------------------------------|--|-------------------------|--|---------------------------------------------------|--|------------------------------|
|  | Do not reuse                     |  | EO sterilization        |  | Warnings and Precautions                          |  | Manufacturer Name Address    |
|  | Used-by date                     |  | CE Symbol               |  | Manufacture Date                                  |  | Consult instructions for use |
|  | Batch Code                       |  | Catalogue Number        |  | Name and Address of European Union Representative |  | Medical device               |
|  | Single sterile barrier system    |  | Keep away from sunlight |  | Keep dry                                          |  | Do not resterilize           |
|  | Do not use if package is damaged |  | Importer                |  |                                                   |  |                              |

【Manufacturer】 Allmed Medical Products Co., Ltd

【Address】 No.18 Qixing Road, Majiadian Town, 443200 Zhijiang City, Hubei Province , CHINA

【European Representative】 Shanghai International Holding Corp. GmbH(Europe)

【Address】 Eiffestrasse 80, 20537 Hamburg, Germany

【Revision Date】 2025-04-01

